berkemann

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Art.-Nr. 08310

1. Produktbeschreibung

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11. Hersteller

Berkemann Hungary Bt.
H-6100 Kiskunfélegyháza
Majsa út 30

www.berkemann.com
product@berkemann.com
Made in Hungary

berkemann

INSTRUCTIONS FOR USE

SPLAYFOOT BANDAGE BY BERKEMANN

Art. No. 08310

1. Product Description

The splayfoot bandage alleviates complaints such as those caused by splay and flat feet. At the same time, the support relieves, protects and supports the bones, muscles, ligaments and fasciae, especially in the area of the transverse arch, thanks to the skin-friendly, compression-generating material, thus promoting mobility. This high-quality product is CE-certified, equipped with silver ions and therefore has an antibacterial effect.

2. Intended Purpose

For support of the transverse arch in cases of discomfort in the forefoot area, particularly in cases of splayfoot and flatfoot.

3. Indications

- Discomfort associated with splayfoot and flatfoot
- Pressure pain in the forefoot area
- Overstrain of the transverse arch

4. Contraindications

Do not use in case of:

- Open wounds in the area of application
- Skin diseases or irritation in the forefoot area
- Known hypersensitivity to any of the materials

If symptoms persist, consult a healthcare professional.

5. Application

1. Clean and dry the foot
2. Pull the bandage over the forefoot
3. Check the fit – the bandage must not constrict or slip
4. May be worn barefoot or inside shoes

6. Wearing Duration

- Wear for 1–2 hours daily at the beginning
- Gradually increase wearing time if well tolerated
- Discontinue use if pain occurs

7. Care Instructions

- Hand wash with mild detergent
- Do not bleach
- Do not dry clean
- Do not tumble dry
- Do not iron
- Do not wring
- Air dry

8. Material

Elastic textile material.
Equipped with silver ions and therefore antibacterial.

9. Scope of Delivery

1 pair of splayfoot bandages

10. Sizes

Select size according to forefoot circumference:

Size	Forefoot Circumference	Article Number	Content
S	17–19 cm	08310 002 630	1 pair
M	20–21 cm	08310 002 640	1 pair
L	22–23 cm	08310 002 650	1 pair
XL	24–25 cm	08310 002 660	1 pair

11. Manufacturer

Berkemann Hungary Bt.
H-6100 Kiskunfélegyháza
Majsa út 30

www.berkemann.com
product@berkemann.com
Made in Hungary

Handwäsche Gentle washing Lavage cycle délicat Programa de lavado delicado	Nicht im Trockner trocknen Do not tumble dry Séchage en tambour impossible No secar en la secadora	Chlor- bleiche nicht möglich Do not bleach Blanchiment ou-chlore impossible No blanquear con cloro	Nicht chemisch reinigen Do not dry clean Nettoyage chimique impossible No someter lavado químico	Nicht bügeln Do not iron Ne pas repasser No planchar