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11. Hersteller

Berkemann Hungary Bt.
H-6100 Kiskunfélegyháza
Majsa út 30

www.berkemann.com
product@berkemann.com
Made in Hungary

INSTRUCTIONS FOR USE

SPLAYFOOT BANDAGE WITH PRESSURE PAD BY BERKEMANN

Art. No. 08320

1. Product Description

The splayfoot bandage alleviates complaints such as those caused by splay and flat feet. In doing so, the support relieves, protects and supports the bones, muscles, ligaments and fasciae due to the skin-friendly, compression-generating material and thus promotes mobility. The additional pad supports the transverse arch from below. This high-quality product is CE-certified, equipped with silver ions and therefore has an antibacterial effect.

2. Intended Purpose

For support of the transverse arch in cases of discomfort in the forefoot area, particularly in cases of splayfoot and flatfoot.

3. Indications

- Discomfort associated with splayfoot and flatfoot
- Pressure pain in the forefoot area
- Overstrain of the transverse arch

4. Contraindications

Do not use in case of:

- Open wounds in the area of application
- Skin diseases or irritation in the forefoot area
- Known hypersensitivity to any of the materials

If symptoms persist, consult a healthcare professional.

5. Application

1. Clean and dry the foot
2. Pull the bandage over the forefoot
3. The integrated pressure pad must be positioned centrally beneath the transverse arch
4. Check the fit – the bandage must not constrict or slip
5. May be worn barefoot or inside shoes

6. Wearing Duration

- Wear for 1–2 hours daily at the beginning
- Gradually increase wearing time if well tolerated
- Discontinue use if pain occurs

7. Care Instructions

- Hand wash with mild detergent
- Do not bleach
- Do not dry clean
- Do not tumble dry
- Do not iron
- Do not wring
- Air dry

8. Material

Elastic textile material with integrated metatarsal pad. The pad can also be removed. Equipped with silver ions and therefore antibacterial.

9. Scope of Delivery

1 pair of splayfoot bandages with pressure pad

10. Sizes

Select size according to forefoot circumference:

Size	Forefoot Circumference	Article Number	Content
S	17–19 cm	08320 002 630	1 pair
M	20–21 cm	08320 002 640	1 pair
L	22–23 cm	08320 002 650	1 pair
XL	24–25 cm	08320 002 660	1 pair

11. Manufacturer

Berkemann Hungary Bt.
H-6100 Kiskunfélegyháza
Majsa út 30

www.berkemann.com
product@berkemann.com
Made in Hungary

 Handwäsche Gentle washing Lavage cycle-délicat Programa de lavado delicado	 Nicht im Trockner trocknen Do not tumble dry Séchage en tambour impossible No sacar en la secadora	 Chlor- bleiche nicht möglich Do not bleach Blanchiment ou Chloro impossible No cloro con cloro	 Nicht chemisch reinigen Do not dry clean Nettoyage chimique impossible No someter lavado químico	 Nicht bügeln Do not iron Ne pas repasser No planchar	 Nicht wringen Do not wring Ne pas tordre No escurir el bulto
---	---	--	--	--	--